

# Skin Grafting Family

The Power of Precision



**Durable. Dependable. Dynamic.**

## Zimmer Biomet Skin Graft Mesher

The Zimmer Biomet Skin Graft Mesher is an easy-to-use skin grafting tool that offers cutters in a variety of expansion ratios. Without the hassle of using multiple machines, five common expansion ratios are provided – 1:1, 1.5:1, 2:1, 3:1, 4:1. When using the Zimmer Biomet Skin Graft Mesher, it's necessary to use the smooth side of the dermacarrier because the cutter determines the ratio. The expansion ratio can be identified on the gear end of the cutter.

The Zimmer Biomet Skin Graft Mesher also features:

- **Convenient Hinge Design** to allow the cutters to be changed quickly or removed for cleaning
- **Built-in Comb** minimizes the chances of grafts sticking and wrapping around the cutters
- **Autoclave Case** makes sterilization and transportation simple
- **Lubeless Gears** help to ensure a smooth performance, time after time

### Skin Graft Mesher Items

Zimmer Biomet Skin Graft Mesher Only <sup>1</sup>	00-7701-000-00
Skin Graft Mesher Complete w/ Four Cutters & Autoclave Case <sup>2</sup>	00-7701-001-00
Skin Graft Mesher w/ Two Cutters <sup>3</sup>	00-7701-002-00
Skin Graft Mesher Autoclave Case	00-7701-003-00
Skin Graft Mesher Ratchet Handle	00-7701-022-00

### Skin Graft Mesher Cutters

1:1 Expansion Ratio	00-7703-010-00
1.5:1 Expansion Ratio	00-7703-015-00
2:1 Expansion Ratio	00-7703-020-00
3:1 Expansion Ratio	00-7703-030-00
4:1 Expansion Ratio	00-7703-040-00

### Skin Graft Mesher Carriers

Dermacarriers Length 8 in. (20.3 cm)	00-7708-000-10
Dermacarriers Length 16 in. (40.6 cm)	00-7716-000-10



Zimmer Biomet Skin Graft Mesher<sup>2</sup>  
00-7701-001-00



MeshGraft II Complete 00-2195-000-00<sup>4</sup>  
MeshGraft II Instrument Only 00-2195-001-00<sup>5</sup>

## MeshGraft™ II Tissue Expansion System

Using the grooved side of the Dermacarriers™, the MeshGraft II System offers four expansion ratios – 1.5:1, 3:1, 6:1, and 9:1.

The MeshGraft II instrument consists of a ratchet handle, continuous feed roller, a cutting roller, and a guidance plateau. The guidance plateau helps ensure proper alignment of the Dermacarrier II Skin Graft Carriers and cutting rollers.

The ratchet handle facilitates proper advancement of the carrier.

### MeshGraft II Carriers

1.5:1 Dermacarriers II Skin Graft Carrier	00-2195-012-00
3:1 Dermacarriers II Skin Graft Carrier	00-2195-013-00
6:1 Dermacarriers II Skin Graft Carrier	00-2195-014-00
9:1 Dermacarriers II Skin Graft Carrier	00-2195-015-00

With a variety of ratios offered by the Zimmer Biomet Meshers, you can reduce clutter by minimizing the amount of equipment needed during a procedure.

## References

1. Includes Mesher, Autoclave Case & Ratchet Handle
2. Includes 1.5:1, 2:1, 3:1 & 4:1 Cutters, Mesher, Autoclave Case and Ratchet Handle
3. Includes 1.5:1 & 3:1 Cutters, Mesher, Autoclave Case and Ratchet Handle
4. Includes Mesher, Autoclave Case, Ratchet Handle, Allen Wrench and Manual
5. Includes Mesher, Ratchet Handle, Allen Wrench and Manual



## CSTAR Service and Repair Programs

Do you have CSTAR protection on all of your skin grafting equipment? All Zimmer Biomet dermatomes are recommended for service every 12 months. Annual electronic calibration checks are strongly recommended. Learn more at [www.service.zimmer.com](http://www.service.zimmer.com).

**The Zimmer Biomet sales force is relied upon in Operating Rooms around the country every day. Being a Zimmer Biomet customer means you're connected to knowledgeable and trained experts.**

**Contact your Zimmer Biomet representative or visit us at [www.surgical.zimmer.com](http://www.surgical.zimmer.com).**



**ZIMMER BIOMET**  
Your progress. Our promise.™

All trademarks are the property of their respective owners. This documentation is intended exclusively for physicians and is not intended for laypersons. Information on the products and procedures contained in this document is of a general nature and does not represent and does not constitute medical advice or recommendations. Because this information does not purport to constitute any diagnostic or therapeutic statement with regard to any individual medical case, each patient must be examined and advised individually, and this document does not replace the need for such examination and/or advice in whole or in part. Please refer to the package inserts for important product information, including, but not limited to, contradictions, warning, precautions, and adverse effects.