


Universal Power System


Maintain Efficiency with
the Power of Modularity

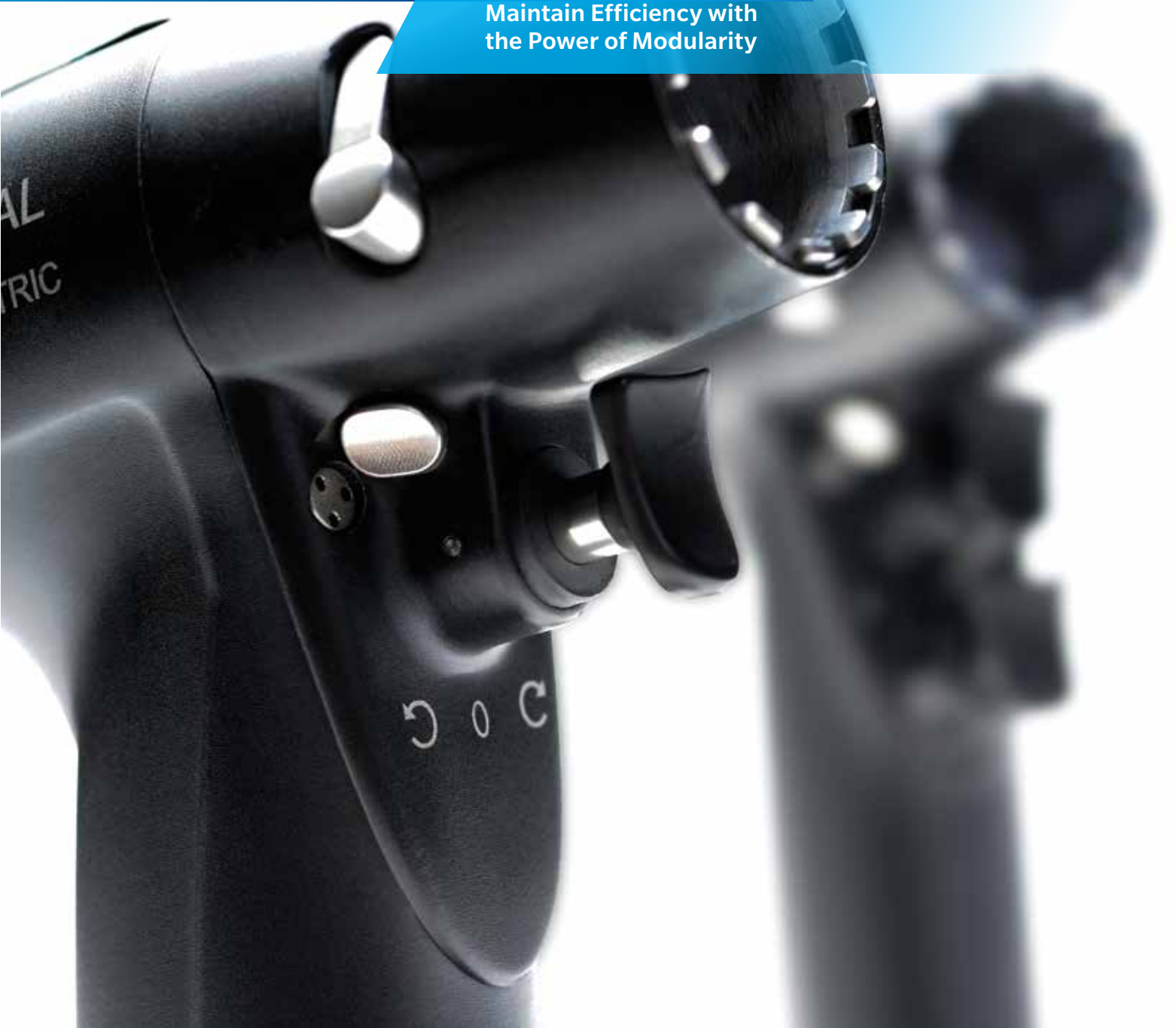

Universal Power System

The All-in-One Surgical Power Instrument

One handpiece does it all: drilling, reaming, and sawing. Our Universal Power System's modular design allows you to configure your surgical power sets for the specific clinical requirements of your operating room. Modularity, combined with an easy-to-use, easy-to-maintain system, may increase your operational efficiencies, all while improving your day-to-day cost of ownership.

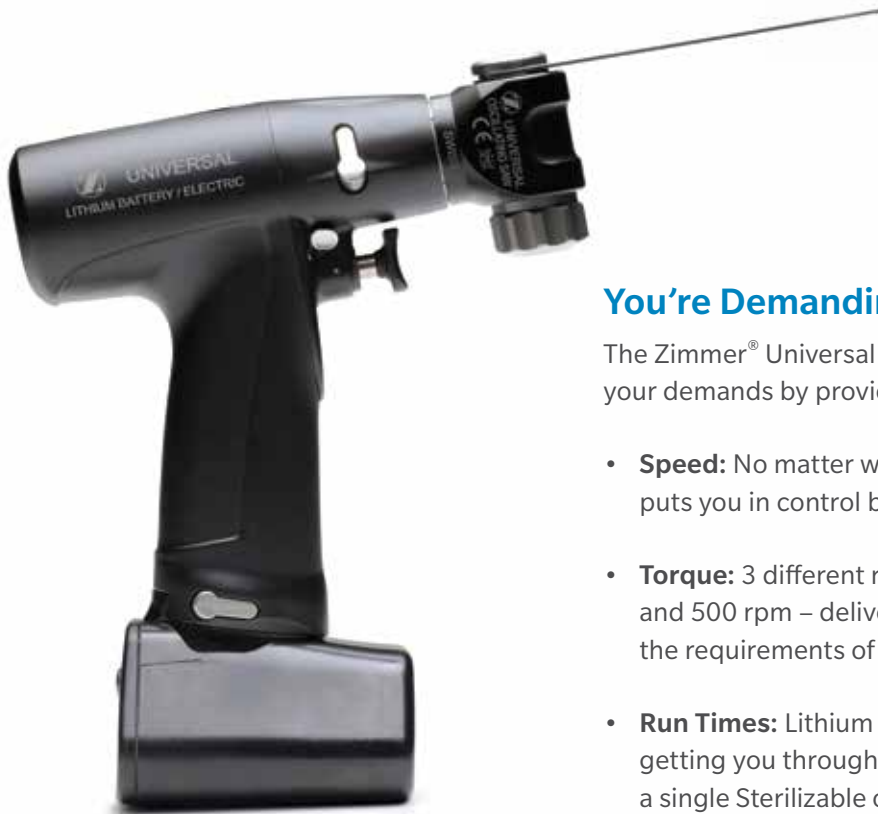

You're Demanding and So Are We

The Zimmer® Universal Power System meets your demands by providing:

- **Speed:** No matter what speed you demand, our handpiece puts you in control by providing variable speed.
- **Torque:** 3 different reaming speeds – 250 rpm, 350 rpm and 500 rpm – deliver the right amount of torque to meet the requirements of your individual case.
- **Run Times:** Lithium lasts longer than nickel based batteries, getting you through an entire procedure with a single Sterilizable or Aseptic battery.
- **Charge Cycles:** Sterilizable – 150/Aseptic – 300 means less batteries to purchase over the life of the set.

Choose Your Power Source

Battery and Electric

The power you demand with a lifespan 2x that of traditional nickel-based technology.



Compact Battery Charger
89-8510-421-00
Accepts Sterilizable
and Aseptic batteries



Sterilizable Battery
89-8510-440-50



Aseptic Battery
89-8510-440-20




Aseptic Battery Housing
89-8510-440-10
Aseptic Battery Funnel
89-8510-440-30




Electronic Power System
89-8510-420-10
Sterilizable Cord
89-8510-420-20


Power Cords

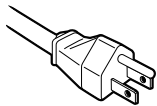


United States
89-8510-421-01

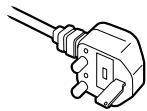

United Kingdom
89-8510-421-02

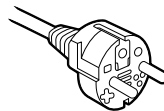

Central Europe
89-8510-421-04

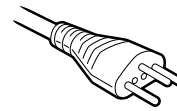

Switzerland
89-8510-421-05

Pneumatic

Power source for surgical power tools utilizes wall or compressed air sources.



Single Air Hose

3 mm 89-8510-456-30
5 mm 89-8510-456-50

Adapter for Single Air Hose

Synthes 89-8510-115-60
Schrader 89-8510-115-81

Double Air Hose - Draeger

3 mm 89-8510-563-00
5 mm 89-8510-565-00

Double Air Hose - Synthes

3m 89-8510-563-01
5m 89-8510-565-01

Washing Cap for Pneumatic Hose – Single 89-8510-455-50

Washing Cap for Pneumatic Hose – Double 89-8510-555-00

Choose Your Handpiece

Lightweight and ergonomic design means reduced hand fatigue for the surgeon as well as balance and control. Modularity eliminates the need for dedicated handpieces, reducing the total number of handpieces and your capital investment.

Battery/Electric Handpieces

Experience the power of simplicity - powered by battery or electric.

Pneumatic Handpiece



Single Trigger Handpiece
89-8507-400-10
One single toggle switch
determines the direction.



Double Trigger Handpiece
89-8507-400-00
Press the bottom trigger for
forward or both triggers for
reverse.



Double Trigger Handpiece
89-8507-405-00
Pneumatic handpieces are
available in double trigger only.

Choose Your Attachments

Our attachments meet the requirements of most implant systems with a singular modular handpiece.

Sawing

Designed to provide speed, precision, and reliable performance during large-bone procedures.



Oscillating Saw

89-8509-450-60

- For knee, hip and trauma procedures
- Rotates to 12 positions in 30 degree increments











Reciprocating Saw

89-8509-451-20

- For knee, hip and trauma procedures
- Rotates to 12 positions in 30 degree increments

Reaming

Provides the torque, speed and precision you need by providing attachments available in 250, 350 and 500 rpm configurations.

		250 rpm	350 rpm	500 rpm
	Zimmer	89-8509-425-70	89-8509-435-70	89-8509-450-70
	Zimmer/Hudson	89-8509-425-80	89-8509-435-80	89-8509-450-80
	Hudson	89-8509-425-40	89-8509-435-40	89-8509-450-40
	1/4" Jacobs Chuck (with key)	89-8509-425-90	89-8509-435-90	89-8509-450-90
	AO	89-8509-425-10	89-8509-435-10	89-8509-450-10
	Dynamic Screw	89-8509-425-65	89-8509-435-65	89-8509-450-65
	Trinkle	89-8509-425-50	89-8509-435-50	89-8509-450-50
	Harris	89-8509-425-30	89-8509-435-30	89-8509-450-30

Drilling

A variety of drilling options are available to accept most drill bit shanks. Options include:



Zimmer/Hudson 1000 rpm

89-8509-415-80



Keyless Chuck 1000 rpm

89-8509-410-90



Small AO 1000 rpm

89-8509-410-20



Pin Driver 2.0 – 4.0 mm

89-8509-451-42



Hudson 1000 rpm

89-8509-415-40



Wire Driver 0.6 – 2.2 mm

89-8509-451-43



1/4" Key Chuck 1000 rpm

89-8509-410-80

5/32" Key Chuck 1000 rpm

89-8509-410-85

Choose Your Attachments (cont.)

Specialty Adapters



Radiolucent Drill Adapter
89-8509-461-20



Drilling/Reaming Adapter
89-8509-412-50



Intramedullary Adapter
89-8509-461-10

Specialty Modular Coupling

To be used in conjunction with Drill/Ream and Intramedullary adapters.



Modular Zimmer Coupling
89-8509-460-70



Modular Small AO Coupling
89-8509-460-20



Modular Harris Coupling
89-8509-460-30



Modular Zimmer/Hudson Coupling
89-8509-460-75



Modular AO Coupling
89-8509-460-10



Modular Key Chuck
89-8509-460-80



Modular Hudson Coupling
89-8509-460-40



Modular Dynamic Screw Coupling
89-8509-460-65



Modular Keyless Chuck
89-8509-460-90



Modular Trinkle Coupling
89-8509-460-50

Choose Your Sterilization Case

Sterilization Cases



Sterilization Case with Lift-Out Tray
89-8510-459-46

The convenient lift-out tray keeps handpieces and all attachments organized and easily accessible. The removable lift-out tray can be placed directly in the automated washer to simplify the washing process.



Trauma Sterilization Case
89-8510-459-43



Pneumatic Sterilization Case
89-8510-459-42

Sterile Tray Lid (Not Pictured) 00-5900-099-00

STABLECUT® Saw Blades

There's more to a surgical blade than the sharpness of the teeth. The stability of the blade under power is critical.

STABLECUT saw blades are designed to keep the teeth on their intended cutting path, intended to give you more control over your technique.

Zimmer Biomet features an extensive selection of blade geometries as part of the STABLECUT family. The portfolio is designed to meet your clinical needs and accommodate the vast array of capital power equipment within the hospital.

Comprehensive Portfolio

Extensive array – No matter the type of orthopaedic surgical technique, the STABLECUT brand aims to offer a saw blade that meets patient specific needs and surgeon specific preferences. Specific Solutions for Specific Joints

Specific Solutions for Specific Joints

There are specific STABLECUT blades recommended for Zimmer Biomet orthopaedic solutions including: Persona® The Personalized Knee System, NexGen® Complete Knee Solution, Vanguard XP® Total Knee System and Oxford® Partial Knee.

STABLECUT blades are designed to accommodate competitive surgical implants.

Choose Your Cutting Tools

Oscillating Saw Blades - for use with Oscillating Saw Attachment (89-8509-450-60)



Thickness (mm)	Usable Length (mm)	Width (mm)	Blade Design	Straight	Flare
0.64	60	19	FLAT	19-0600-64A-F1	-
0.76	24	9	TOOTHSET	09-0240-76A-T1	-
	34	9	TOOTHSET	09-0340-76A-T1	-
0.89	90	25	FLAT	25-0900-89A-F1	25-0900-89A-L1
	90	19	FLAT	19-0900-89A-F1	-
	90	13	FLAT	13-0900-89A-F1	-
1.00	110	25	FLAT	25-1101-00A-F1	-
	105	25	FLAT	25-1051-00A-F1	-
	90	25	FLAT	25-0901-00A-F1	-
	105	19	FLAT	19-1051-00A-F1	-
	90	19	FLAT	19-0901-00A-F1	21-0901-00A-L1
	90	13	FLAT	13-0901-00A-F1	-
1.19	65	30	TOOTHSET	30-0651-19A-TL	-
	100	25	GTS COARSE	-	25-1001-19A-VC
	90	25	GTS COARSE	25-0901-19A-GC	-
	105	20	GTS COARSE	20-1051-19A-GC	20-1051-19A-VC
	100	20	GTS COARSE	-	20-1001-19A-VC
	90	20	GTS COARSE	20-0901-19A-GC	20-0901-19A-VC
	80	20	GTS COARSE	20-0801-19A-GC	-
	105	13	FLAT	13-1051-19A-F1	-
	90	13	DIAMOND	13-0901-19A-D1	-



Choose Your Cutting Tools (cont.)

Oscillating Saw Blades (cont.) - for use with Oscillating Saw Attachment (89-8509-450-60) (cont.)



Thickness (mm)	Usable Length (mm)	Width (mm)	Blade Design	Straight	Flare
1.27	110	25	GTS	25-1101-27A-G1	-
	105	25	GTS	25-1051-27A-G1	-
	100	25	GTS COARSE	-	25-1001-27A-VC
	90	25	GTS COARSE	25-0901-27A-GC	-
	105	20	GTS COARSE	20-1051-27A-GC	20-1051-27A-VC
	100	20	GTS COARSE	-	20-1001-27A-VC
	90	20	GTS COARSE	20-0901-27A-GC	20-0901-27A-VC
	80	20	GTS COARSE	20-0801-27A-GC	-
	60	19	FLAT	19-0601-27A-F1	-
	105	13	FLAT	13-1051-27A-F1	-
	90	13	DIAMOND	13-0901-27A-D1	-
	75	13	GTS	13-0751-27A-G1	-
	1.37	110	25	GTS	25-1101-37A-G1
105		25	GTS	25-1051-37A-G1	-
100		25	GTS COARSE	-	25-1001-37A-VC
90		25	GTS COARSE	25-0901-37A-GC	-
105		20	GTS COARSE	20-1051-37A-GC	20-1051-37A-VC
100		20	GTS COARSE	-	20-1001-37A-VC
90		20	GTS COARSE	20-0901-37A-GC	20-0901-37A-VC
80		20	GTS COARSE	20-0801-37A-GC	-
105		13	FLAT	13-1051-37A-F1	-
90		13	FLAT	13-0901-37A-F1	-
1.47	65	35	TOOTHSET	35-0651-47A-TL	-
	90	25	GTS	25-0901-47A-B1*	25-0901-47A-V1
	90	19	GTS	19-0901-47A-G1	21-0901-47A-V1
	90	13	FLAT	13-0901-47A-F1	-

*Ribbed Blade

Reciprocating Saw Blades - for use with Reciprocating Saw Attachment (89-8509-451-20)



Blade Design	Usable Length (mm)	Width (mm)	Thickness (mm)	Part Number
Single Sided	90	12	1.19	12-0901-19U-R1
Single Sided	70	10	1.19	10-0701-19U-R1
Double Sided	76	13	0.89	13-0760-89U-R1
Double Sided	76	12	1.00	12-0761-00U-R1
Double Sided	76	12	1.00/1.19	12-0761-19U-R1
Keel Cut - Cemented	90	13	3.55	51-KEEL-CMU-R1
Keel Cut - Cementless	90	12	2.87	47-KEEL-CLU-R1



Radiolucent Drill Bits - for use with Radiolucent Drill Attachment (89-8509-461-20)

Diameter	Length (mm)	Part Number
2.5	148	89-8508-041-25
2.7	148	89-8508-041-27
3.2	148	89-8508-041-32
3.3	148	89-8508-041-33
3.5	148	89-8508-041-35
4.0	148	89-8508-041-40
4.2	148	89-8508-041-42
4.3	148	89-8508-041-43
4.5	148	89-8508-041-45
3.2	106	89-8508-042-32
4.0	106	89-8508-042-40





CSTAR™ Service and Repair Programs

CSTAR service programs offer a broad range of services to keep your Universal Power Equipment at peak condition over its designed lifetime. Zimmer Biomet has highly trained technicians available for full service support. Additionally, daily use and maintenance training is available for hospital personnel upon request at no additional cost. For access to the support of trusted experts, visit service.zimmer.com.

The Zimmer Biomet sales force is relied upon in Operating Rooms around the country every day. Being a Zimmer Biomet customer means you're connected to knowledgeable and trained experts. For more information, contact your Zimmer Biomet representative.



ZIMMER BIOMET
Your progress. Our promise.™

All trademarks are the property of their respective owners. This documentation is intended exclusively for physicians and is not intended for laypersons. Information on the products and procedures contained in this document is of a general nature and does not represent and does not constitute medical advice or recommendations. Because this information does not purport to constitute any diagnostic or therapeutic statement with regard to any individual medical case, each patient must be examined and advised individually, and this document does not replace the need for such examination and/or advice in whole or in part. Please refer to the package inserts for important product information, including, but not limited to, contradictions, warning, precautions, and adverse effects.